




This is how you use Yellowone HandsafeTM


Attention:

Please remove the safety sticker on the front of the product before use.

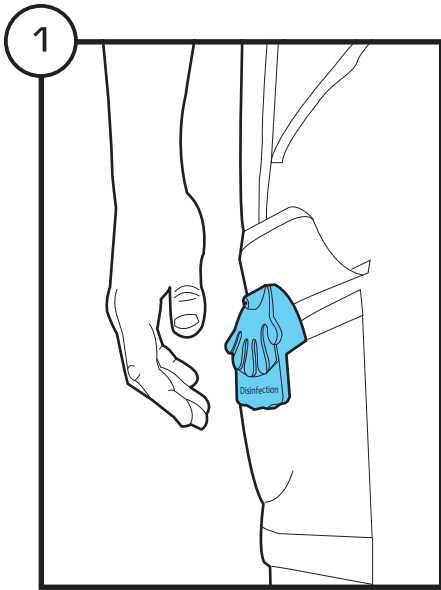

Yellowone HandsafeTM dispenser may be worn using the intergral garment clip.


Place it inside your coat/pant pocket, belt or bag by sliding the dispenser top along the edge of the garment to the opposite side for "locking" it in place by the barbs inside.

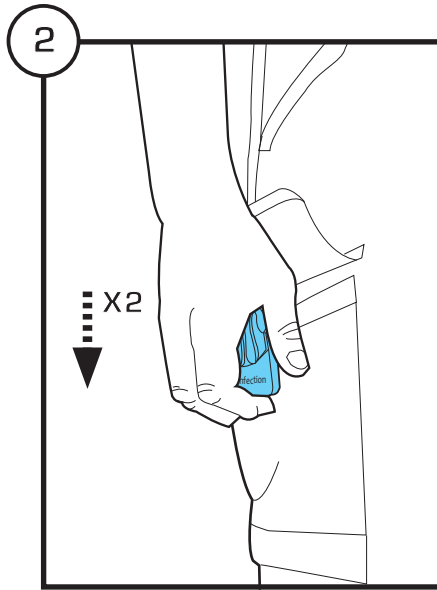

You operate the dispenser with one hand.

Grab your fingers onto the clip part and press rapidly the dispenser top with your hand palm to dispense the alcohol-based gel.

Use at least 2 pumps (= 3 ml) each time to perform correct hand hygiene.

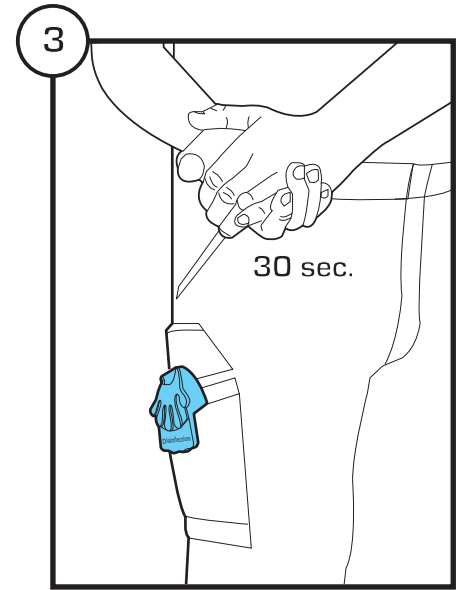

Rub hands and wrist carefully until dry (about 30 sec).

Online video instruction:
<https://www.youtube.com/watch?v=S9P5MWxDPfI>

Be careful not to contaminate the dispenser during use. The dispenser must be cleaned once a day and kept clean and dry when not in use (e.g. end of shift/not yet opened dispensers). The alcohol-based gel used in the dispenser contains 85% alcohol. The product is CEI approved in Denmark and also holds EN 1500 and EN 12054 tests, which is safe to use and works effectively against bacteria, fungi and viruses.

